

EyeMax 2 Phototherapy Protection Mask

Our latex-free EyeMax 2 provides optimum comfort and protection for patients during phototherapy treatments. The one piece, wrap-around design has two attachment points that can be adjusted independently; this prevents unwanted movement and allows for a perfect fit. The ergonomic eye-protection pad was specially designed to prevent light leakage.



ULTIMATE PROTECTION




A separate light-blocking material is included inside the eye covering portion of the eye mask to **shield retinas from harmful damage**.

GENTLE ON SKIN




Gentle material was created with the fragility of patients in mind with **no adhesives to irritate skin**.

SECURE FIT

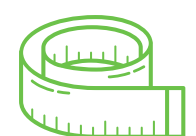



Three different sizes that are each fully adjustable by **two separate velcro points**.

LIGHTWEIGHT MATERIAL

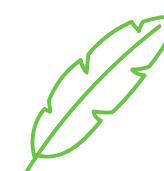

Light is able to pass through the thin material to allow **maximum skin exposure**.

EyeMax 2 Phototherapy Protection Mask

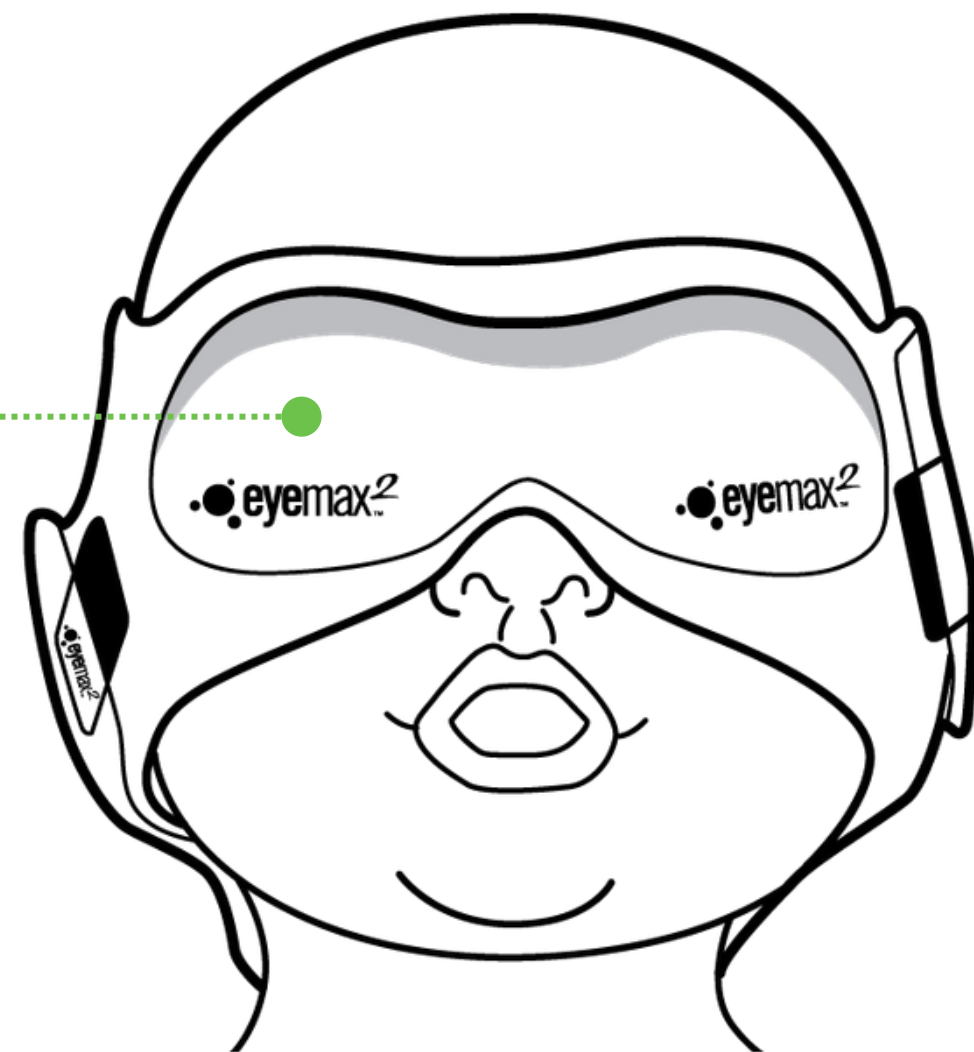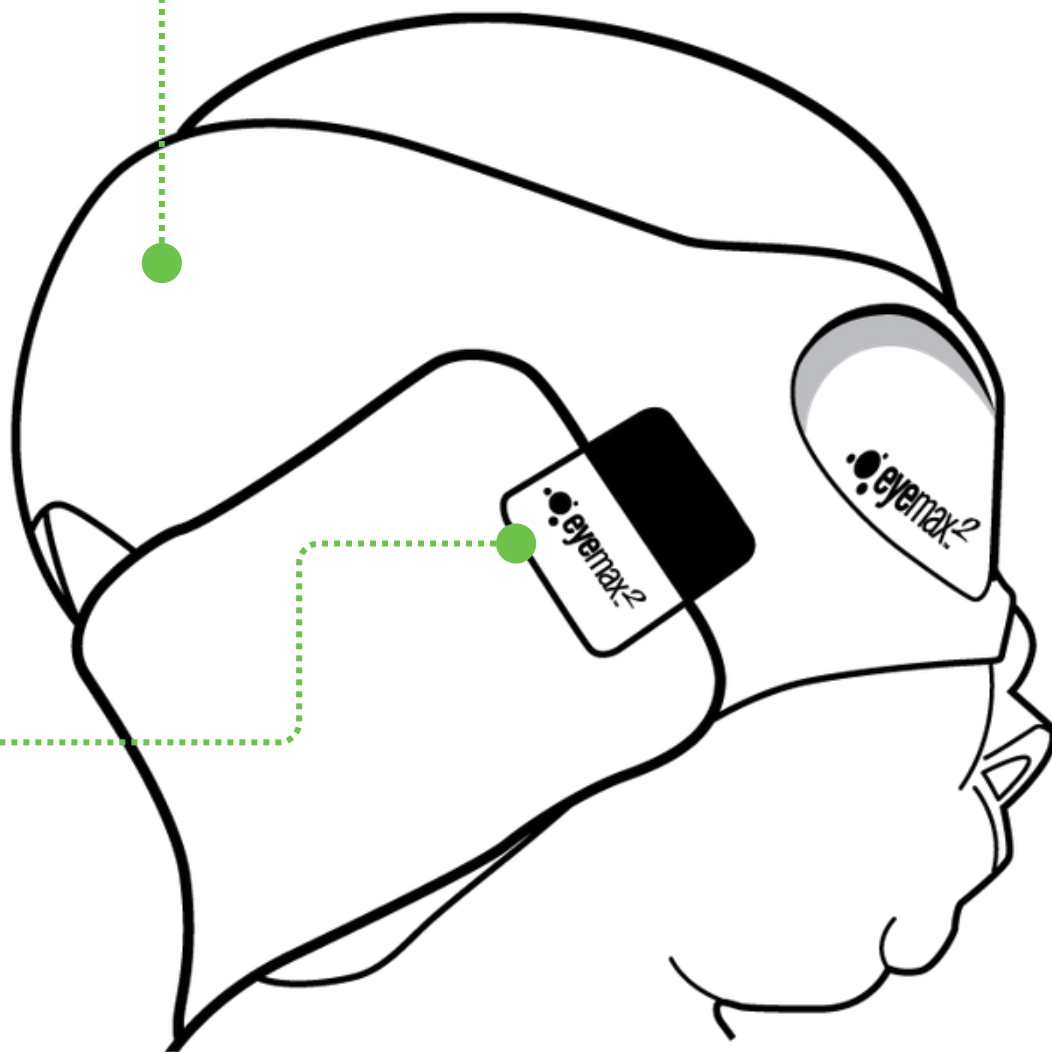

1 VELCRO STRAPPING

Adjustable band to create the perfect fit

2 EYE PADS

Added material to cover eyes for increased protection

3 LATEX FREE

Helps to prevent irritant contact dermatitis

READY TO SHIP

All three sizes are in stock and ready to protect your patients.



SUPPORT

Maxtec customers are equipped with continued support for any sales and technical questions through a variety of channels, including a live chat option on our website.



EyeMax 2 Size Guide

REGULAR (12.6"-14.9")
(Occipital Frontal Circumference)



PREMIE (10.4"-12.6")
(Occipital Frontal Circumference)

MICRO (7.87"-10.4")
(Occipital Frontal Circumference)



PART NUMBERS

EyeMax 2 Regular (20 pack)	_____	R300P01
EyeMax 2 Premie (20 pack)	_____	R300P02
EyeMax 2 Micro (20 pack)	_____	R300P03

How To Use The EyeMax 2

INDICATIONS FOR USE:

The EyeMax2 provides neonatal eye protection during ultraviolet (UV) phototherapy treatment of jaundice. The EyeMax2 conforms with EN 170:2002 Clause 5.2 (Scale Number 2-5) for spectral transmittance as appropriate for the indication for use.

INSTRUCTIONS FOR USE:

1. Measure Occipital-Frontal Circumference (OFC) of the patient.
2. Select the appropriate EyeMax2 size based on the OFC. See chart for appropriate sizing.
3. Disconnect the Velcro.
4. Position the eye pads over the eyes.
5. Pass the lower strap under the small of the neck and attach velcro.
6. Secure the Velcro of both straps and position the EyeMax2 as shown.
7. Make sure the strap completely covers the ears.
8. If the eye patch rides up away from the nose, tighten the lower strap.
9. If the eye patch rides down towards the nose and mouth, repeat application process beginning with step 1.

SYMBOL GUIDE

The following symbols and safety labels are found on the EyeMax2:



Medical Device



RX Only

PRECAUTIONS:

CAUTION: Improper EyeMax2 sizing may lead to patient airway obstruction or inadequate eye protection. The Eyemax2 should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

-  **DO NOT** tug on the EyeMax2 to reposition it. A premature infant's skin is very delicate and can easily rupture. Avoid obstructing the nose. Check regularly to ensure the EyeMax2 is not blocking the patient's airway. The EyeMax2 is not a bandage. Cover any existing skin abrasions with appropriate bandages before use.
-  **DO NOT** over tighten - causing excessive pressure on the eyes. Make sure the strap is covering the ear, not hooked over the ear, and that the ear is not folded over. Replace the EyeMax2 with a new one if it becomes over-stretched. Reuse of this device could cause cross-contamination or product malfunctions.

**WATCH A STEP-BY-STEP
DEMONSTRATION HERE**



**LEARN MORE, REQUEST A
DEMO, OR GET A QUOTE**